Health o meter®

P r o f e s s i o n a l

PROPLUS® 4560/4700

USER INSTRUCTIONS INSTRUCCIONES DE USO MODE D'EMPLOI

**ISO13485:2003
CERTIFIED**

**HEALTH O METER® PROFESSIONAL
PELSTAR LLC**

CORPORATE HEADQUARTERS

11800 SOUTH AUSTIN AVENUE

ALSIP, IL 60803 USA

TOLL FREE: 1 (800) 815-6615

TELEPHONE: 1 (708) 377-0600

FACSIMILE: 1 (708) 377-0601

WWW.HOMSCALES.COM

PATENTS 7550682B2 AND D508655 (US); PATENTS 0149984/1-8 AND PENDING EU);
Patents ZL200480031711X and 200430004551.2 (PRC) © Pelstar, LLC 2011

Rev. 20110701

DIGITAL WHEELCHAIR PLATFORM SCALE USER INSTRUCTIONS

Thank you for your purchase of this Health o meter® Professional product. Please read this manual carefully and keep it for easy reference or training.

TABLE OF CONTENTS

ENGLISH.....	2
ESPAÑOL.....	14
FRANÇAIS-.....	26

TABLE OF CONTENTS

CAUTIONS AND WARNINGS.....	3
SPECIFICATIONS	3
CERTIFICATIONS / CONNECTIVITY / SERVICE	4
ASSEMBLY INSTRUCTIONS	5
SET UP / BATTERIES	6
QUICK START INSTRUCTIONS	7
MAINTENANCE / CALIBRATION.....	9
TROUBLESHOOTING	10
DETAILED VIEW OF SCALE	11
PARTS FOR MAINTENANCE	12
WARRANTY	13

PLEASE REGISTER YOUR SCALE FOR WARRANTY COVERAGE AT:
<http://www.homscales.com/customers/registration.aspx>

CAUTIONS AND WARNINGS

To prevent injury and damage to your scale, please follow these instructions very carefully.

- Do not transport the scale while the platform is loaded. Unplug the adapter and store both the adapter and cord before moving the scale. Be sure the scale is tilted enough to assure it is rolling properly on its wheels when it is moved for cleaning or transported from one area to another.
- For accurate weighing the scale must be placed on a flat, stable surface.
- For accurate weighing, verify proper operation according to the procedure described in this manual before each use. Do not exceed recommended weight limit of 1,000lb/454kg for this scale.
- Do not use in the presence of flammable materials.
- Operating at other voltages and frequencies than specified could damage the equipment.
- If the “LOW BAT” indicator activates, for accurate weighing, replace the batteries or connect the scale to a power source, using the power adapter, as soon as possible, depending on use—see Connectivity section.

SPECIFICATIONS

GENERAL

The ProPlus® 4650/4700 Wheelchair Platform Digital Scale for Patients uses highly sophisticated microprocessor technology. Each precision scale is designed to provide accurate, reliable and repeatable weight measurements and features that make the weighing process simple, fast and convenient.

These scales are set up to use motion-sensing weighing technology, to determine the actual weight of a moving patient. The scales may be changed to measure live weight (see the ProPlus® Display Module User Manual for instructions on how to change the scale settings).

Weight is displayed in pounds. The scale features a remote display which may be held by the operator or placed on a table, desk, wall or other flat surface for privacy and convenience in use. The 4650/4700 has 2 wheels built in for easy mobility. The scale can be operated using a power adapter (included) or by 6-D cell batteries (not included), depending on use—see Connectivity section, below.

4650/4700 SPECIFICATIONS

Capacity and Resolution	4650/4700 ,: Max 1,000 lb / 0.2lb, 454 kg / 0.1 kg	
Power Requirements	120V-240V AC, 50-60Hz - 9V DC adapter, ADAPTER MODEL ADPT31 (INCLUDED) or 6 D-cell batteries (not included)	
Environmental	Operating temperatures: 50°F to 95°F (10°C to 35°C) Storage temperatures 32°F to 122°F (0°C to 50°C) Maximum Humidity: 85% RH	
Physical Dimensions	Platform Size	Ramp Size
4650	35 ¾ (w) 32 ¼ (d)	10” x 32-1/4”
4700	43 (w) x 42 (d)	12-3/4 x 42

CERTIFICATIONS / CONNECTIVITY

CERTIFICATION DESCRIPTIONS

The logo consists of the text "ISO13485:2003" on the top line and "CERTIFIED" on the bottom line, both in a bold, sans-serif font, enclosed within a rectangular border.

Pelstar, LLC has been officially certified as a manufacturer of Health o meter[®] Professional medical devices to be in compliance with stringent legal requirements. The Pelstar, LLC quality assurance system covers the development, production, sales and service of medical scales and measuring systems.

4650/4700 SCALE CONNECTIVITY INFORMATION

These scales are designed to connect to a computer, monitor or other devices via USB or serial ports, to allow reliable transmission of weight, height and BMI from the scale. This connectivity helps close a source of potential medical errors caused by incorrectly copying and then recording weight information taken. This scale is EMR-EHR compatible. For more information regarding how to connect this scale to devices please see the ProPlus[®] Display Module User Manual for instructions or contact Customer Service at one of the numbers below.

CUSTOMER SERVICE INFORMATION

Our fully staffed, in-house customer service team is equipped to answer any questions or concerns that may arise.

Telephone: 1(800) 815-6615

Facsimile: 1 (708)377-0601

HomProCS@homScales.com

4650/4700 ASSEMBLY INSTRUCTIONS

Before Assembly (4650/4700):

The ProPlus® 4650/4700 Wheelchair Patient Digital Platform Scale is shipped in one carton. Carefully inspect the carton for shipping damage before unpacking. If damage is found, contact Customer Service. The following information details what you will find inside the main carton.

Remove each assembly from the carton and unwrap packing material carefully, to prevent scratching the unit's parts. Set the carton aside, for storage. To avoid damaging the scale parts when unpacking it, **do not use a box cutter, knife, scissors or any sharp object to open the protective inner packaging.**

Parts List:

- (1) Base / Platform Assembly (with 2 ramps, handle, wheels attached)
- (1) User information
- (1) Box Containing:
 - (1) Display Assembly with attached Load Cell Connector Cable
 - (1) Small Box Containing:
 - (1) Power Adapter
 - (4) Feet

Assembling a 4650/4700 Scale:

This scale takes only minor assembly.

1. Place scale base on a flat, level, dry floor—place upside down carefully.
2. Thread the 4 feet into the underside of the base. Each foot should be completely hand tightened and then backed out 1-½ full turns. This allows for proper load cell operation for accuracy of the scale.
3. Carefully tilt the scale back to an upright position on the floor with care to watch the ramp—it may flip down when the scale is tipped back. **DO NOT DROP THE SCALE.** Adjust the feet, to level the scale. Be certain that the feet are NOT completely tightened into the base (see Step 2, above).
4. Open the scale to operational position.
5. Connect the load cell cable connector to the load cell port on the scale base.
6. Connect the power adapter cord to the power port in the display, if the adapter will be used. The cord will be outside of the display, as shown in Figure 1. It is optional to run the cord through the channel in the back of the display, if you prefer. To accomplish this, remove the 4 screws (A) in the back with a Phillips screwdriver, remove the cover, run the power cord (B) through the channel, and then replace the cover, secure it with the 4 screws and tighten.
7. The scale is now assembled. Plug the scale adapter into a power source and / or insert batteries—see the batteries section / Figure 2 for more information regarding batteries.

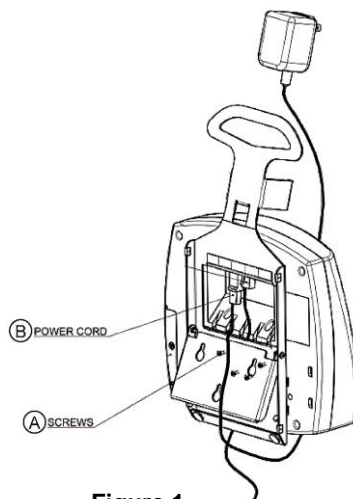
Figure 1

SET UP / BATTERIES

ProPlus[®] scales are shipped with the Menu Lock function enabled. The Menu Lock function disables certain buttons on the scale's keypad, preventing unauthorized users from making changes to your scale settings. The Menu Lock can be bypassed temporarily to allow you to customize your scale with the functionality you prefer. Please refer to the ProPlus[®] Display Module User Manual for instructions on how to bypass the Menu Lock.

Preparing the Scale for Use

1. Remove protective plastic films covering the keypad and display.
2. Place batteries in the battery holder (see "Batteries" information in Figure 2, below).
3. Plug the scale's power adapter into the power source.
4. Be sure all objects are removed from the scale's platform, and then press the **ON/OFF** or **I/O** button to turn the scale on. The display will show "Health o meter ProPlus" and then "0.0 lb".

Note: To adjust the display backlight and/or contrast, please refer to the ProPlus[®] Display Module User Manual.

Note: The scale will always default to the settings and units last used.

5. When you disconnect the scale's power adapter from the power source, the scale will switch to battery power, if batteries are installed. **Note:** If the set up procedure failed, refer to the troubleshooting instructions. If the problem is not corrected, refer to qualified service personnel.

Replacing / Inserting Batteries

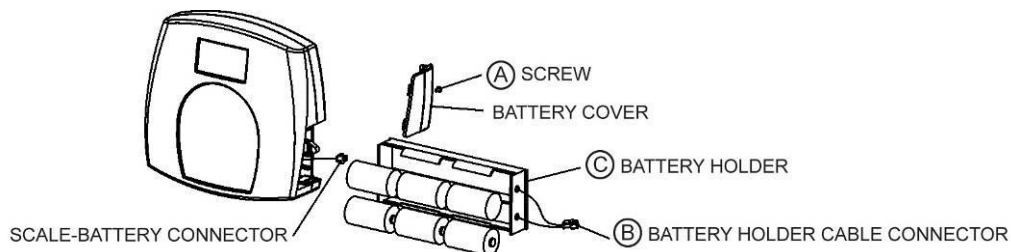
Figure 2 - Batteries

1. Unplug the scale.
2. Remove the battery cover from the display assembly—refer to Figure 2 (A) above.
3. Disconnect battery holder cable connector from the scale-battery connector (B).
4. Carefully remove the battery holder (C) by sliding it out of the display assembly.
5. Replace the batteries with new ones. We recommend EVEREADY Energizer[®] e2[™] batteries.
6. Carefully slide the battery holder (C) into the display assembly.
7. Plug the battery holder cable connector to the scale-battery connector (B).
8. Attach the battery cover to the display assembly and install the screw (A).

Warning: If the scale will not be used for some time, remove batteries to avoid a safety hazard. Disposal of batteries should be done according to the national, regional or local regulations which apply to you for disposal of batteries.

SET UP / TRANSPORT

Preparing the Scale for Transport

The scale can easily be folded for movement from one room to another and includes pins on one ramp, to carry the indicator. To prepare the scale for transport, follow these steps:

1. Fold both ramps--left and right (A) onto the platform while the scale is on the floor (see Figure 3).
2. Place the indicator (B) onto the pins on one of the folded ramps by sliding the slots that are on the back of the indicator onto the raised pins.

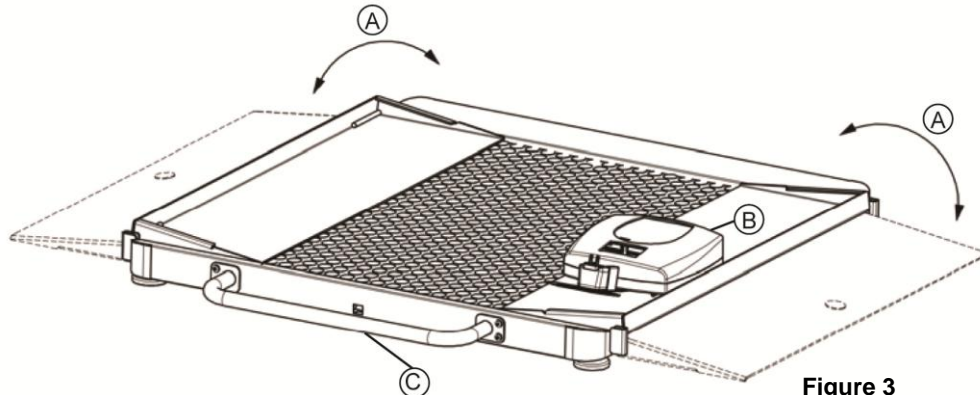
Figure 3

3. Lift the scale onto its side by pulling up on the handle (C). The scale will now be ready to transport, rolling on its wheels (see Figure 4).

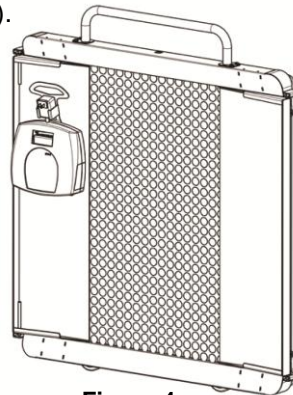
Figure 4

QUICK START INSTRUCTIONS

Additional, complete instructions regarding all of the display and keypad functions and menus can be found in the ProPlus® Display Module User Manual included with this scale.

NOTE: This scale will always default to the settings and units last used, when it is turned on.

WEIGHING A PATIENT

1. Make sure there is no object on the weighing platform.
2. Press the **ON/OFF** or **I/O** button to turn the scale on.
3. Wait until "0.0 lb" and "ZERO" on the left side of the display appear.
4. Roll the patient onto the scale. The scale will calculate the patient's weight but the display will not show a weight until the stable weight of the person is determined. Depending on the movement of the patient on the scale, it may take several seconds for the scale to lock onto the weight.
5. Once a weight has been displayed, while the patient is still on the scale, we recommend performing a second weighing by pressing the REWEIGH button.
6. After the weight measurement has been taken and is recorded, roll the patient roll off the scale.
7. Press the **ON/OFF** or **I/O** button to turn the scale off.

QUICK START INSTRUCTIONS (CONTINUED)

CALCULATING BMI

Body Mass Index is another name for Quetelet Index, a formula that is a convenient and reliable indicator of obesity. **NOTE:** The scale will not calculate BMI for a patient that is 26.4 lb or less. If a height is not entered for the patient within 30 seconds of pressing the BMI button, the scale will return to the normal weighing mode.

1. Complete steps 1 to 4 above for "Weighing A Patient".
2. After obtaining the patient's weight on the scale, press the BMI (2) button while the scale is on.
3. The scale will prompt you to enter the patient's height. Use the keypad to enter the height.
NOTE: When weighing in kilograms, the height must be entered in 0.1 centimeter increments.
4. Once height is keyed in, press **ENTER**.
5. The display will show the patient's **BMI** (Body Mass Index).
6. Press **EXIT** to return to the normal weighing mode of the scale.

TARE FUNCTION

When using the scale, the weight of an object such as a wheelchair can be subtracted from the total weight to determine the weight of the patient alone. The Tare Function automatically performs this subtraction. ProPlus® scales allow tare weight to be entered manually by using the keypad or automatically, as outlined in this section.

AUTOMATIC (PUSH BUTTON TARE)

1. Wait until "0.0 lb" and "ZERO" appear on the left side of the display.
2. With all weight off the scale, press the **ON/OFF** or **I/O** button to turn on the scale.
3. Place the wheelchair on the scale. The display will show a value for the wheelchair's weight.
4. Press the TARE (9) button. The scale will zero out and the word "TARE" will appear in the upper right side of the display to indicate there is a tare value for this patient's wheelchair.
5. Remove the weight from the scale. The display will show a negative weight equal to the weight of the wheelchair, once it is removed from the scale.
6. Place the patient and wheelchair on the scale. The scale will automatically deduct the tared weight of the wheelchair from the gross weight of the patient and wheelchair together.
7. The weight of the patient will appear on the scale.
8. The tared value is stored in memory until changed, cleared or the scale is turned off.
9. To remove the Tare, press the TARE (9) button again. The word "TARE" will disappear from the display and the tare value will be deleted from memory. You can also press the EXIT button. The scale will resume normal operation.

MANUAL (KEYBOARD OR KEYPAD TARE)

1. Make sure there is no object on the weighing platform.
2. Press the **ON/OFF** or **I/O** button to turn on the scale.
3. Wait until "0.0lb" and "ZERO" appear on the left side of the display.
4. Press the **TARE (9) button**. The user will be prompted to enter the tare value.
5. Use the keypad to enter the weight of the shoes or other object and then press **ENTER**.
6. The value entered will show as a negative value on the display.
7. Place the patient and the tared object on the scale. The scale will automatically deduct the tared weight, as entered, from the gross weight of the patient with the tared object.
8. The weight of the patient will appear on the scale.
9. The tared value is stored in memory until changed, cleared or the scale is turned off.
10. To remove the Tare, press the **TARE (9) button** again. The word "TARE" will disappear from the display and the tare value will be deleted from memory. You can also press the **EXIT** button. The scale will resume normal operation.

REMOVING THE TARE VALUE

Press the **TARE (9) button** again to delete a tare value from memory. The word "TARE" will disappear from the display and the tare value will be deleted from memory. You can also press the **EXIT** button. The scale will resume normal operation.

MAINTENANCE

The following pages provide instructions for maintenance, cleaning, troubleshooting, and operator replaceable parts for the ProPlus[®] 4650/4700 scale. Maintenance operations other than those described in this manual should be performed by qualified service personnel.

MAINTENANCE

Before first use and after periods of non-use, check the scale for proper operation and function. If the scale does not operate correctly, refer to qualified service personnel.

1. Check overall appearance of the total scale for any obvious damage, wear and tear.
2. Inspect the power adapter for cord cracking or fraying or for broken or bent prongs.

CLEANING

Proper care and cleaning is essential to ensure a long life of accurate and effective operation.

Disconnect the scale from the adapter power source.

1. Clean all external surfaces with a clean damp cloth or tissue. Mild antimicrobial or antibacterial soap and water solution may be used. Dry with a clean soft cloth.
2. Do not immerse the scale into cleaning or other liquid solution. Do not gas or steam sterilize the scale, nor place any part of it through an autoclave.
3. Do not use Isopropyl Alcohol or other solutions to clean the display surface.
4. Do not use abrasive cleaners.

4650/4700 CALIBRATION

This scale has been factory calibrated and does not require calibration prior to use. For maintenance, the scale can be calibrated. Calibration of your ProPlus[®] scale is performed using pounds (lb).


Display	Operator Action
“Enter load weight: “400.0 lb”	1. Make sure the display is turned off, then press and hold the HOLD/RELEASE button while pressing the ON/OFF or I/O button.
“Enter load weight: “400.0lb”	2. Using the ▲▼ keys and/or the numeric keypad, set the calibration load to at least half the capacity of the scale (for the best results and greater accuracy, use the full capacity of the scale).
Zero calibration: Please clear the scale	3. Make sure there is nothing on the scale, then press the ENTER button.
Zero calibration: Please wait	4. Please wait 1-3 seconds, without touching or shaking the scale, to allow the Zero Calibration to finish.
Weight calibration: Put: XXX.X	5. No action required by operator.
Weight calibration Please wait	6. Load the scale with the required weight and press the ENTER button. Please wait 1-3 seconds, without touching or shaking the scale, to allow the Calibration to finish.
Factor Update... “Factor OK!” or “Factor Fail!”	7. If calibration is successful, the display will show “Factor OK!”. If the calibration process failed, it will show “Factor Fail!”.
Rebooting... Please wait	8. If successful, the display will begin to reboot.
Health o meter ProPlus	9. The display will return to the start up message.
Health o meter ProPlus	10. Please wait until the scale resumes normal operation and then remove the weight from the scale.

TROUBLESHOOTING

Refer to the following instructions to check and correct any failure before contacting Customer Service.

CORRECTIVE ACTION	POSSIBLE CAUSE	SYMPTOM
<ol style="list-style-type: none"> 1. Replace batteries 2. Use a different outlet 3. Replace adapter 	<ol style="list-style-type: none"> 1. Dead Battery 2. Faulty electrical outlet 3. Bad power supply 	Scale does not turn on
<ol style="list-style-type: none"> 1. Remove interfering object from the scale 2. Ask the patient to get off the scale, zero the scale and begin weighing process again 3. Place the scale on a level floor and begin weighing process again 4. Check weight with known weight value 5. Place the item to be tared on the scale. Press REWEIGH. Once the weight of the item is displayed, press TARE. Place the patient and the tared item back on the scale. Press REWEIGH again 	<ol style="list-style-type: none"> 1. External object interfering with the scale 2. The display did not show "0.0" before weighing 3. Scale is not placed on a level floor 4. Scale is out of calibration 5. Improper Tare 	Questionable weight or the scale does not zero
Ask the patient to stand still or you can change to live weight setting	The patient is not standing still	Weighing is performed but the display shows "weigh" and "reweigh" every few seconds; the weighing process takes too long and no weight is displayed.
Remove the excess weight and use the scale according to its limits	The load on the scale exceeds the capacity (1,000 lb or 454 kg)	The display shows "Overload" message
Replace batteries according to instructions	The batteries are empty	The display shows "LOW BAT" message
Check load cell cable connection at the display and platform assembly ports. If the problem is not corrected, refer to qualified service personnel to replace the defective load cell	There is a problem with one or more load cells or the load cell cable is disconnected.	The display shows "Load Cell Error" message

DETAILED VIEW OF 4650/4700 SCALE

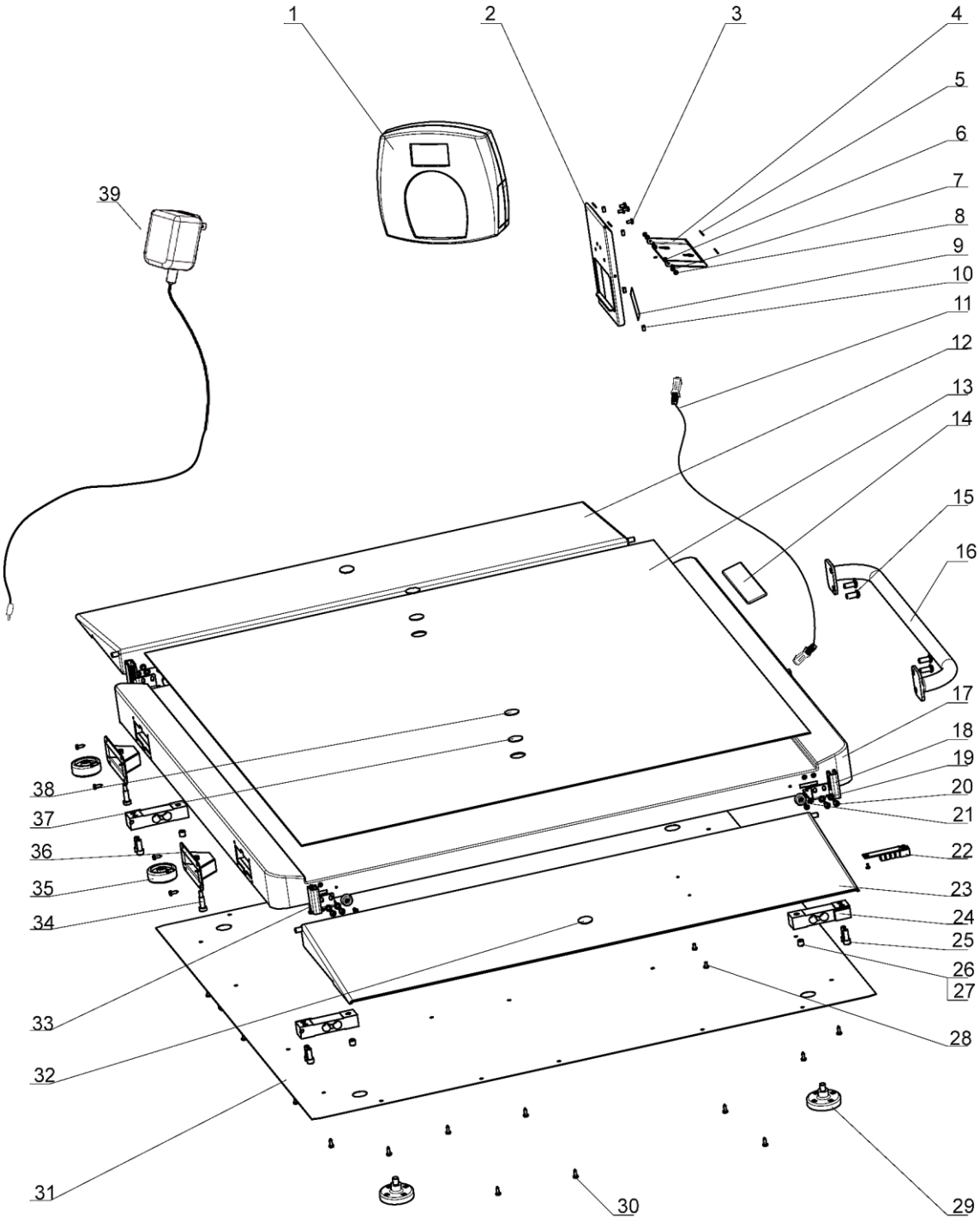4650/ 4700 PARTS FOR MAINTENANCE

Field Replaceable Parts			
Qty.	Description	Part No.	Key No.*
1	DISPLAY ASSEMBLY	B1109401-0	1
1	MAIN BOARD TO JUNCTION BOX LOAD CELL CABLE	B2033701-0	11
2	RIGHT RAMP BRACKET	B3823601-0	18
1	CONNECTOR BOARD	B2138901-0	22
4	LOAD CELL	B1304202-0	24
4	FOOT ASSEMBLY	B2266701-0	29
2	LEFT RAMP BRACKET	B3823701-0	33
1	POWER ADAPTER 120V-240V	ADPT31	39
1	INCLUDES ADAPTER, DISPLAY ASSY, 4 LOAD CELLS, CONNECTOR BOARD, 4 FOOT ASSEMBLY	PROKIT	
Below listed items are for reference only (refer to detailed view on page 11)			
1	DISPLAY STAND		2
4	PHILLIPS PAN HEAD SCREW, M4		3
1	DISPLAY STAND FOOT		4
4	NON SKID PADS		5
2	SELF LOCK NUT, M3		6
2	WASHER, M3		7
2	SCREW, M3x10		8
1	INPUT / OUTPUT POSTS LABEL		9
4	BUMPER		10
1	RAMP #1		12
1	MAT		13
1	HEALTH O METER LABEL		14
4	SCREW, M8x19		15
1	HANDLE		16
1	PLATFORM ASSEMBLY		17
16	WASHER, M5		19
16	SCREW, M5x10		20
4	RAMP MAT		21
1	RAMP		23
8	SOCKET HEAD CAP SCREW 5/16-18x¾		25
4	SCREW, M10x10		26
4	LOAD CELL SAFETY SCREW		27
2	SCREW, M4x6		28
26	SHEET METAL SCREW, #8x½		30
1	BOTTOM COVER		31
2	SCOTCH MATE HOOK		32
2	SHOULDER BOLT		34
2	WHEEL		35
2	WHEEL CLIP ASSEMBLY		36
2	DOUBLE STICK TAPE		37
2	SCOTCH MATE LOOP		38

* See page 11 for a detailed view of parts by number.

WARRANTY

3- YEAR LIMITED WARRANTY

What does the Warranty Cover?

Health o meter® ProPlus® scales are warranted from date of purchase against defects of materials or in workmanship for a period of three (3) years. If product fails to function properly, return the product, freight prepaid and properly packed to Pelstar, LLC (see "To Get Warranty Service", below, for instructions). If the manufacturer determines that a defect of material or in workmanship exists, the customer's sole remedy will be repair or replacement of the scale at no charge. Replacement will be made with a new or remanufactured product or component. If the product is no longer available, replacement may be made with a similar product of equal or greater value. All parts including repaired and replaced parts are covered only for the original warranty period.

Who is Covered?

The original purchaser of the product must have proof of purchase to receive warranty service. Please save your invoice or receipt. Pelstar dealers or retail stores selling Pelstar products do not have the right to alter, or modify or any way change the terms and conditions of this warranty.

What is Excluded?

Your warranty does not cover normal wear of parts or damage resulting from any of the following: negligent use or misuse of the product, use on improper voltage or current, use contrary to the operating instructions, abuse including tampering, damage in transit, or unauthorized repair or alternations. Further, the warranty does not cover Acts of God, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights, and you may also have other rights that vary from country to country, state to state, province to province or jurisdiction to jurisdiction.

To get Warranty Service make sure you keep your sales receipt or document showing proof of purchase. Call (+1) 800-638-3722 or (+1) 708-377-0600 to receive a return authorization (RA) number, which must be included on the return label. Attach your proof of purchase to your defective product along with your name, address, daytime telephone number and description of the problem. Carefully package the product and send with shipping and insurance prepaid to:

Pelstar, LLC
Attention R/A# _____
Repair Department
11800 South Austin Avenue
Alsip, IL 60803

If your scale is not covered by warranty, or has been damaged, an estimate of repair costs or replacement costs will be provided to you for approval, prior to servicing or replacing.

PRODUCT LIFE: 10 YEARS / 100,000 CYCLES

PELSTAR, LLC
11800 South Austin Avenue • Alsip, IL 60803 • USA
www.homscal.es.com

ProPlus® is a registered trademark of Pelstar, LLC. Health o meter® is a registered trademark of Sunbeam Products, Inc. used under license. Health o meter® Professional products are manufactured, designed and owned by Pelstar, LLC. We reserve the right to improve, enhance or modify Health o meter® Professional product matures or specifications without notice.

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Pelstar LLC